

STOP BLEEDING, INSTANTLY.



Axiostat Haemostatic Dressing - Military variant, is specially designed to be used in battlefield and conflict zones. It comes in camouflaged, rugged metal pouch packing for easy carrying and single-handed use.

Battlefield
proven
technology

axiostat[®]
stops bleeding, instantly

MIL88 - 100% Chitosan Haemostatic Dressing




www.axiobio.com

100% CHITOSAN HAEMOSTATIC DRESSING

3 yr shelf-life

FDA CE 2460

How to use Axiostat?

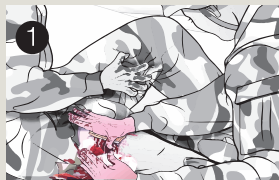

Apply Axiostat with pressure until dressing sticks to wound



Once applied, do not lift or remove Axiostat, not even to check hemostasis. Apply field dressing to secure Axiostat in position



To remove Axiostat, irrigate with saline and gently peel it off



Painless Removal




To treat gunshot wounds multiple Axiostat may be used as shown

Indication:

Axiostat® is a sterile, single use, non-absorbable haemostatic dressing intended to be used for temporary control of bleeding wounds.

For more info, Visit: www.axiobio.com/military

How it works?

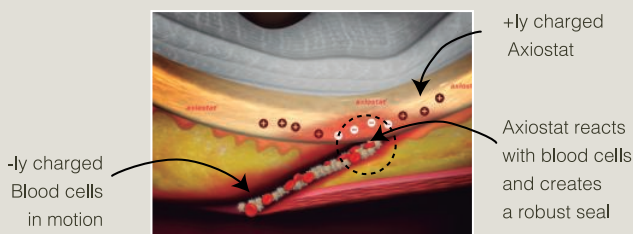

Recommended For:

- Arterial & Venous Bleeding
- Gunshot Wounds
- Blast injuries
- Puncture or Stab Wounds
- Cuts, Lacerations




MIL88 : 8cmx8cm

- ☑ Sterile, ready to use
- ☑ Hypoallergenic, non-exothermic, non-pyrogenic
- ☑ Active on all sides
- ☑ Can be cut, folded, stuffed into deep wounds
- ☑ Protects wound from infection

Clinical Data

Time in minutes

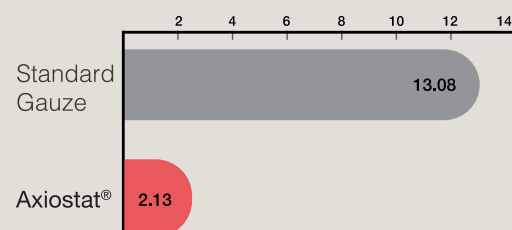

Clinical Study on Trauma patients
Published in Indian Journal of Emergency Medicine
IJEM VOL.2 No.2 JULY-DEC 2016



AXIO BIOSOLUTIONS PRIVATE LIMITED

Plot 18, Gujarat Pharma TechnoPark,
Sari Matoda, Sanand Taluka,
Ahmedabad-382220, Gujarat, India.

Customer care: **+91 8860786067**

Email: info@axiobio.com

www.axiobio.com

twitter.com/axiobio

facebook.com/axiobio

